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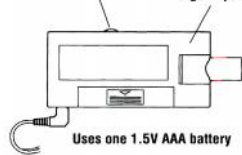
On/Off switch and volume control wheel

High impact ABS case

Uses one 1.5V AAA battery

Multi-element high sensitivity microphone

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3 YEAR WARRANTY

Sonic Technology Products, Inc. warrants to the original purchaser that your SuperEar® will be free of defects in materials or workmanship for a period of 3 years from the date of purchase. Warranty is void if defects are caused by negligence, abuse, misuse or damage caused by water or heat.

Returns made under this Warranty must be accompanied by \$5.00 to cover the handling and return postage cost.

This Warranty gives you specific legal rights and you may have other rights which may vary from state to state.

Should it be necessary to send your SuperEar® in for service under the terms of this Warranty, return it in a well-packaged carton and ship postage prepaid.

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Instructions for Use

1. Remove the battery compartment cover by lightly depressing the cover with your thumb and slide it in the direction of the arrow.
2. Insert the AAA 1.5V battery. Be sure to match the positive (+) and negative (-) ends to those indicated on the battery cavity. Alkaline batteries will deliver 2-3 times the operational life of lesser batteries.
3. Slide battery compartment cover back into locked position.
4. Insert the earphone connector into the jack on the left side of the amplifier.
5. Be sure the amplifier switch is in the "off" position before putting on the earphones. Adjust headband to fit comfortably.
6. Slowly turn the volume control knob on the right side of the amplifier to the desired level of amplification.
7. Attach the pocket clip to the bottom of the amplifier using the screw provided. Do not over-tighten. Can be used on pocket, belt or other gear.

NOTE: When using your Super Ear® outdoors, protect the amplifier from exposure to moisture. This device is not waterproof.

User Tip

1. This amplifier has a maximum gain of over 50 dB. At approximately 3/4 open on the volume control, full volume is achieved. If you turn the volume control all the way up there is no additional real sound gain, BUT may result in an annoying feedback noise. Simply turn the volume control down to eliminate feedback.
2. If headphones not in use are placed near the amplifier while it is turned on, feedback squeal will result. **Always** turn your Super Ear® off before you remove the headphones.

Specifications

- Designed to provide up to 50+ decibels of ambient sound amplification. Maximum output of 107dB.
- Frequency response 100 to 14000 hertz.
- Solid state microelectronic circuitry requires only a single 1.5AAA alkaline battery for up to 30 hours of operation. Battery included.
- Premium stereo headphones deliver clear, crisp sound particularly well in the human voice range. The foam ear pads are easily replaceable.
- Operating temperature range 0°C min, 55°C max.
- Relative humidity operating range at 20°C/95% RH.

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©Sonic Technology 2015