	FR	DE	NL	EN
①	Chargement type C/USB	USB-C wiederaufladbar	USB-C oplaadbaar	USB-C rechargeable
②	Volume -	Lautstärke -	Volume -	Vol -
③	ON/OFF Modes	ON/OFF Modi	AAN/UIT Modi	ON/OFF Functions
④	Volume +	+	+	Vol +
⑤	Voyant	Leuchtanzeige	Indicatielampje	Indicator light
⑥	Micro	Mikro	Micro	Micro
⑦	Indication L/R	Links/Rechts-Anzeige	Links/Rechts aanduiding	Left/Right indication

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FR

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SERENITY" SOUND AMPLIFIER

EN

Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

A. SAFETY INSTRUCTIONS

1. The device is intended exclusively for private and not for commercial use. Only use this device as describe in this instruction manual.
2. Do not handle the unit or plug with wet hands.
3. When disconnecting the cable, always pull at the plug; do not pull on the cable itself.
4. Never try to repair it by yourself. It must be repaired by the manufacturer or its service agent or a similarly qualified person in order to avoid an electric shock.
5. Do not use outdoors.
6. Do not let the power cord on the edge of a table or work surface, and prevent it comes into contact with hot surfaces.
7. Keep it away from heat sources such as radiators, to avoid deformation of plastic parts.
8. Do not use coarse or abrasive sponges/cloths or steel wool to clean the appliance.
9. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

B. CONTRAINDICATIONS

- The device must not be used by people suffering from occasional or chronic ear infections, congenital ear defects or deafness.
- Do not use in case of migraines, vertigo, ear pain or tinnitus.
- Do not use in case of ear secretions.
- Do not use in case of allergy.
- Keep out of reach of children - this is not a toy! Small parts can be swallowed!

C. RECHARGING THE BATTERY

This device is equipped with a built-in battery. Before first use, fully charge the battery (charging time 2h30). You can charge this device with a USB adapter (not included) or by connecting it to your computer's USB port (adapter specifications: 5V-1.5A) by connecting the USB cable to the ① port on the device.

You can charge this device using a USB adapter (not supplied) or by connecting it to your computer's USB port (adapter specifications: 5V-1.5A).

The LED ⑤ on the main unit lights up orange when the device is charging, and green when the device is fully charged.

Remember to recharge your device regularly (about every 3 months) to prevent the battery from discharging completely and damaging its capacity.

D. HOW TO USE IT

1) Choice of tips

- Select the tip of your choice
- Push the earmold membrane outward.
- Attach the earmold firmly to the sound amplifier.

2) Getting started

- Differentiate between the two sides of the sound amplifier: L for left and R for right (7) .
- Place the amplifier around your neck.
- Insert the headphones into your ears.
- Press and hold the (3) button for 3 seconds to turn on the unit.
- The indicator light (5) will flash green.

Note: There may be a crackling sound before the hearing instrument amplifier is properly connected. This crackling sound can be used as a reference to determine if the hearing instrument is properly attached. If the hearing instrument emits a crackling sound, check to see if it is properly attached. If it is, but the crackling continues, replace it with another set of earmolds. When the hearing instrument is not in use, turn it off immediately.

3) Volume adjustment

- Press the (4) button to increase the volume
- Press the (2) button to decrease the volume
- A beep is emitted when the volume is increased or decreased
- 3 beeps are emitted when you are at maximum/minimum volume

Note: It's normal to hear high-pitched whistling or buzzing when the hearing amplifier is at a high volume level and doesn't fit the ear well due to feedback from the microphone. If this happens, turn down the volume and adjust the hearing amplifier in the ear. Also check that you are wearing the correct size earmolds to ensure a good fit in the ear canal, as they can also induce slight feedback if they don't fit properly in the ear.

4) Modes

- Press the (3) button to select one of 2 modes
- Mode 1 standard: 1 beep
- Mode 2 noise reduction: 2 beeps

5) Switching off

- Press the (3) button for 3 seconds to switch off the unit.
- The (5) LED flashes orange and then goes out.

Note: using the sound amplifier may take a little getting used to. At first, the surrounding noise may disturb you, but you'll gradually get used to it.

E. CARE AND MAINTENANCE

Clean the product only with a dry or slightly damp, lint-free cloth.
Do not use abrasive cleaners to clean the unit.

F. FEATURES

- Material: ABS
- Power supply: Li-po 100mAh 3.7V rechargeable battery (USB charging cable included, USB adapter not included)
- Sound amplification: 35±3dB
- Frequencies: 250 to 5000 Hz
- Accessories included: 3 x 2 tips of different diameters, cleaning brush

- Autonomy on full charge: 80h
- Charging time: 2h30

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

Precision and precautions about the battery use

Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.

- It is important to limit the battery consumption and to respect the following rules:
- Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
- Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
- Used battery has to be removed.
- Also, remove the battery if you're not using the apparel for a long time.
- Never try to short-circuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.
- For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.

Importé par / Importiert von / Geïmporteerd door / Imported by :
CAPTELEC, 59170 Croix - FRANCE

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